

INSTRUCTIONS ANAL TEST

This is a reliable test for chlamydia and gonorrhoea

Please take your time to read the instructions. It shouldn't be difficult, but please follow the instructions carefully. This way, errors between




1. Put the tube in a cup so that the liquid does not flow out. Open the tube and take the cotton swab. Do not touch the cotton.




2. Find yourself a comfortable position to bring the swab easily into your anus



3. Bring the cotton swab about 2-3 cm into the anus and turn the cotton swab around 5 seconds around in the anus.

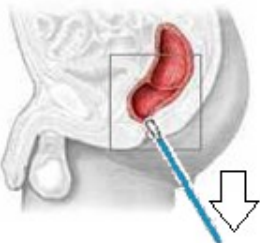

4. Carefully remove the cotton swab. Don't touch the part that has been into the anus. Any defecation on the swab is no obstacle for the analysis

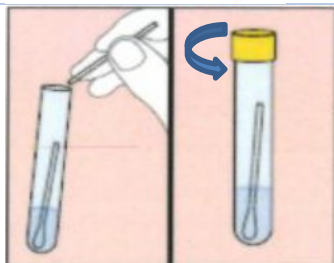

5. Place the cotton swab back in the tube with the liquid. Break the stick at the black mark and close the tube.



6. Put the tube into the plastic safety bag.

Close and seal the safety bag only when all tests are inside.